



WESTMINSTER
INTERNATIONAL LTD

Excellence through Service
Protecting your world




fire



safety



security

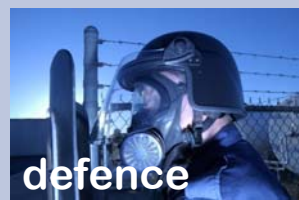

defence

QuickTox Drugs Abuse Dip Card

QuickTox is a multiple drugs of abuse dip card that allows for fast and straight-forward testing of samples. It is capable of testing for up to 11 drugs or 9 drugs and 4 adulterants simultaneously. With its flat design and clear cap, QuickTox can be photographed without fear of contamination of the tester's desk or bench.

Key features

- ▶ Simultaneous detection of multiple drugs;
- ▶ Choice of configurations from 4-12 drugs;
- ▶ Built-in adulterant testing available;
- ▶ Rapid results;
- ▶ Tests activate in 10 seconds;
- ▶ Results are ready in 5 minutes;
- ▶ 1 hour result stability;
- ▶ Eliminates pipetting;
- ▶ Minimises sample handling;
- ▶ Procedural control;
- ▶ 18-month shelf life;
- ▶ FDA cleared.



Test for the following:

Drugs


- ▶ Amphetamine;
- ▶ Barbiturates;
- ▶ Benzodiazepines;
- ▶ Cocaine;
- ▶ Marijuana (THC);
- ▶ Methadone;
- ▶ Methamphetamine;
- ▶ Opiates;
- ▶ Oxycodone;
- ▶ PCP;
- ▶ Tricyclic Antidepressants.

Adulteration Parameters


- ▶ Creatinine;
- ▶ Nitrite;
- ▶ pH;
- ▶ Oxidizer.

THE PROCEDURE


- Dip device up to the **tips of the arrows** for ten seconds and recap immediately.

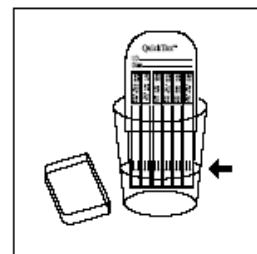

- Read results in 5 minutes. Results are stable up to 1 hour.

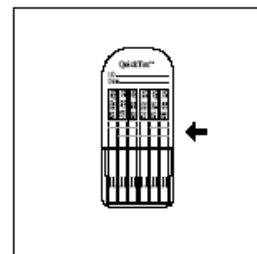

- Results Interpretation:
Control (C) band must be visible for valid test.
- The test is **positive** if there is no test band present
- The test is **negative** if test band is present.

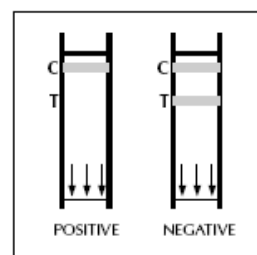

WI Code: 10411/1

Westminster International Ltd,
Westminster House,
Blacklocks Hill,
Banbury,
Oxfordshire OX17 2BS,
United Kingdom.
T: +44 (0)1295 756300
F: +44 (0)1295 756302
E: info@wi-ltd.com
W: www.wi-ltd.com

The Association of
Police and Public
Security Suppliers



The Defence
Manufacturers
Association



QuickTox Drugs Abuse Dip Card

Cut Off Levels

Amphetamine (AMP)	500 & 1,000	Ng/ml
Barbiturates (BAR)	300	Ng/ml
Benzodiazepines (BZO)	300	Ng/ml
Cocaine (COC)	300	Ng/ml
Marijuana (THC)	50	Ng/ml
Methadone (MTD)	300	Ng/ml
Methamphetamine (MET)	500 & 1,000	Ng/ml
Opiates (OPI)	300 & 2000	Ng/ml
Oxycodone (OXY)	300	Ng/ml
Phencyclidine (PCP)	25	Ng/ml
Tricyclic Antidepressants (TCA)	1,000	Ng/ml

WI Code: 10411/1

Westminster International Ltd,
Westminster House,
Blacklocks Hill,
Banbury,
Oxfordshire OX17 2BS,
United Kingdom.
T: +44 (0)1295 756300
F: +44 (0)1295 756302
E: info@wi-ltd.com
W: www.wi-ltd.com

The Association of
Police and Public
Security Suppliers



The Defence
Manufacturers
Association



© 2008 Westminster
International Ltd.
All rights reserved.